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INSTRUCTIONS FOR USE  
MOBILUS LITE  
4-WHEEL ALUMINIUM ROLLATOR  
VCBK222B  
Edition EJ 02.2022

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**PLEASE READ WHOLE INSTRUCTION  
CAREFULLY BEFORE USE**

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Foldable four-wheel rollator, equipped with lockable brakes, allow for navigating any surface; anatomically fitted handles from polyethylene; frame made of aluminium aerofoil sections protected by a non-toxic paint partially anodized.

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1. Handles.
2. Brake lever with locking rotation of the wheels.
3. Seat
4. Zawias rozporowy
5. Height adjustment of handles
6. Mounting arm rear wheel screw
7. Rear wheel
8. Belt safety
9. Bag for small items
10. Front wheel fork
11. Mounting arm front wheel screw
12. Front wheel

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Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

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Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

- The product is packed In folder position.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer.
- Do not use the rollator until all parts are properly connected.
- Unfolding rollator should begin unfolded anchor hinge by pressing both hand seat frame till the moment when You can hear a click. (Illustration 6.)

- Then unscrew the screws attaching the shoulder of the rear wheels. Lower the rear wheels so that you can tighten the screws in the holes of the arms of wheels. Make sure that the screws are correct screwed in.
- Then unscrew the screws attaching the shoulder of the front wheels. Lower the front wheels so that you can tighten the screws in the holes of the arms of wheels. Make sure that the screws are correct screwed in.

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#### FOLDED:

- To facilitate the transport, the rollator can be folded.
- Pull the handle located in the middle of the seat, causing the fold of the rollator cross frame .

- Additionally, you can reduce the size of the rollator by folding wheels arms.
- it is necessary to unscrew the screws fixing the wheel arm, raising the arms to the position shown in illustration 9.
- Tighten screw without wheels arm. Make sure the screws are screwed in.

#### BRAKE SYSTEM

- The rollator is equipped with multi-functional brake. The brake handle is located under the rollator handle. It can operate as a standard brake as well as parking brake
- To use the standard brake, lift up the brake handle.
- To use a parking brake, push down the brake handle to the point where you can hear the distinctive sound of inclusion brake (You

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should use in that same time both the right and left brake).

- You should use in that same time both the right and left brake.

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#### OVERCOMING OBSTACLES (THRESHOLD, CURBS)

- In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle. Holding the handle, tilt the rollator in rear and lightly keep by the foot one of the footer located at the rear wheel, which will raise the front wheels up.
- drive the rear wheels to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

#### CARE AND MAINTENANCE

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the support (wheels, clamping screws, locks).
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the wheelchair.

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- Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the rollator with pressure or steam aggregates!

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#### TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Before each use	Control of wheels, brakes,	The brakes can worn out, check the condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	wheel axles	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant.

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#### Repair

- In case of reparation, please contact the producer.

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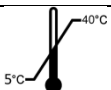
#### Customer service

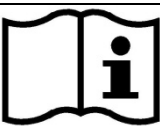
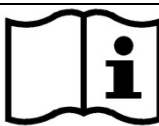
- If you have any questions or need help, contact the producer.

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