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**PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE**

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Four wheel aluminium rollator with footrest , equipped with two fixed rear wheels with brake and locking the wheels and the two front wheels swivel. It lets you easily navigate through any surface, inside and outside the building. Handles anatomical made of polyethylene, aluminium frame.

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1. Handles.
2. Brake lever with locking rotation of the wheels.
3. Height adjustment screw of handles.
4. Rollator frame.
5. Levels hinge of stretcher.
6. Brake.
7. Rear wheel.
8. Backrest.
9. Side frame hinge.
10. Seat .
11. Basket for small items.
12. Footrest.
13. Steering wheel fork.
14. Front wheel.

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Elements that should be inside the packaging:

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Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

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Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

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- The product is packed In folder position.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer.

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Uchwyty mocowania oparcia / Mounting bracket backrest

- Do not use the rollator until all parts are properly connected.
- Unfolding rollator should begin unfolded the front frame and rear frame, then lock the levels hinge of stretcher. (Illustration 6)

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After removing all the security film can unfolded footrest and adjust the seat. (Illustration 7.)

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- In the holes of the side frames should be mounted handle (Illustration 8). In the hole from the inner side of the side frame insert the screw, while the from outer side on the screw should put the

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washer, and then tighten the knob. (Illustration 9.)

- Backrest should be installed in designated for it handle located on the side frame rollator (locking tabs located on the rail must snap into the holes localized in handle).
- After installing the backrest, pull it to make sure it is properly secured.

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- Basket is mounted under the seat.
- Handle to mounting the basket must be on the outer sides of the seat
- In order to fold rollator should follow the opposite, to that of the scheme unfolded..

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- Before using the rollator, refer to this manual and keep it for future reference.
- At standstill and the use of seat should apply the locking rotation of the wheels both rear wheels.
- The stand will not replace the wheelchair, the seat is used only for temporary resting or transport the patient on a short distance.
- The rollator is used only to provide rest to one user. Do not stand on it..
- The rollator should be used on even surfaces at an

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CHANGE POSITION BACKREST, UNFOLD THE FOOTREST

- Depending on additional function that is to act rollator should be set backrest.
- In a situation where the seat is intended only for temporary resting, when the patient moves independently without a caretaker backrest should be positioned as shown in Illustration 12, on the opposite side of handles.
- In a situation when the seat is designed to make transport over short distances Patient backrest should be placed so that figure as presents 13 on the side of handles.

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- To remove the backrest of the handles, simultaneously press the two latches located on the mounting brackets back, and then pull it up. (Illustration 14.)

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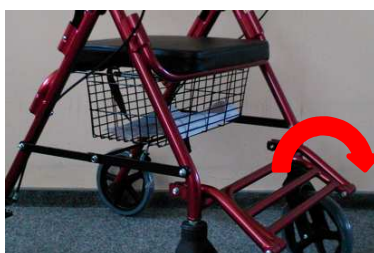
HEIGHT ADJUSTMENT HANDLES.

- Handles should be located at a height that will provide comfort and a sense of security for the user. It is important to ensure that both hands are positioned at the same height.
- To change the height of the handles unscrew the knob for fixing handles, remove washers, the screws and depending on the needs extend or shorten the frame handles.
- After adjusting the appropriate height put the screw on the inner side of the side frame , while the outer side on the screw put pad, and then tighten the knob. (Illustration 9.)
- Make sure that the handle is properly tightened, so that it prevents movement of the handles.

FOLDING:

- In order to fold rollator for ease of transport, you must first block rotation of the wheels
- Then it is necessary to remove the basket for small items, to fold footrest.
- Pick up the seat, pull upward levels hinge of stretcher, then fold the rollator according to move levels hinge of stretcher. (Illustration 18.)
- When moving the rollator, always hold the side frame of rollator.

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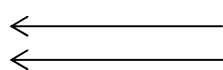
UNFOLD:

- To unfold rollator slide outward side frames, then lock the levels hinge of stretcher.
- Before each use, check the operation of the braking system. Squeeze the brake handles to check if the brake pads are right adhere wheels and stop the rollator.
- Press the handle down to activate the locking rotation of the wheels and check if it works properly.

ADJUSTMENT OF BRAKING

- To adjust the brake force at first loosen the nut is located under the adjusting screw just above the wheel brake.

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BRAKE SYSTEM

- The rollator is equipped with multi-functional brake. The brake handle is located under the rollator handle. It can operate as a standard brake as well as parking brake
- To use the standard brake, lift up the brake handle.
- To use a parking brake, push down the brake handle to the point where you can hear the distinctive sound of inclusion brake (You should use in that same time both the right and left brake).
- You should use in that same time both the right and left brake.

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OVERCOMING OBSTACLES (THRESHOLD, CURBS)

- In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle. Holding the handle, tilt the rollator in rear raise the front wheels up.
- Then drive the rear wheels to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

CARE AND MAINTENANCE

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the support (wheels, clamping screws, locks).
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the wheelchair.
- Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the rollator with pressure or steam aggregates!

TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Before each use	Control of wheels, brakes,	The brakes can worn out, check the

		condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	wheel axles	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant.

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Repair

- In case of reparation, please contact the producer.

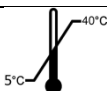
Customer service

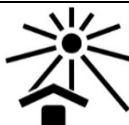
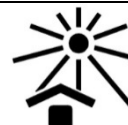
- If you have any questions or need help, contact the producer.

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