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Dane kontaktowe

XIAMEN SENYANG CO.,LTD
Address: No.189-1,Tongji South Road, Jimei District
Xiamen, 361000 Fujian P.R. China

SUNGO Europe B. V.
Address: Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Importer

mdh Sp. z o.o.
Adres: ul. Maratońska 104, 94-007 Łódź, Polska
tel. +48 42 674 83 84, fax. +48 42 636 52 21
www.mdh.pl
www.viteacare.com

English version on next page

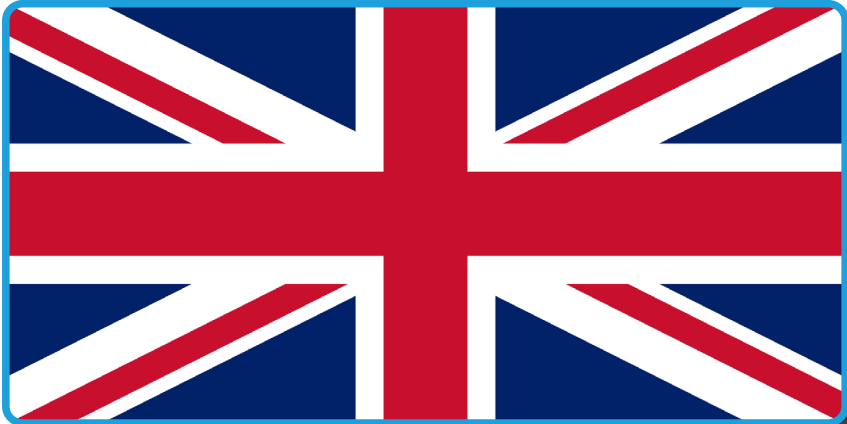
Table of contents

Introduction from mdh Sp. z o.o.....	20
Elevator construction and technical parameters.....	21
Technical specifications.....	22
Indications and contraindications	22
Indications.....	22
Contraindications	22
Assembly	23
List of parts.....	23
Scheme of detachable mains connection	23
Seat assembly	24
Usage	26
Patient transport to an lift - simulation	27
Patient transport to lift - different surfaces	27
Short distance lift driving	28
Lift seat height adjustment.....	28
User and patient safety	29
Maintenance operations	30
Repair	30
Symbols	31
Contact information	33
Importer.....	33

Introduction from mdh Sp. z o.o.

Thank you for choosing our product. We firmly believe that it will meet your expectations. Please read this document before using it for the first time. These instructions are meant to provide you with the necessary information on safe use of the device. Correct compliance with the recommendations will significantly increase the durability and aesthetics of the product. We would like to point out that we are constantly trying to improve the quality of our products, which may result in minor changes that are not included in this manual.

Electric lifts are mainly assistive devices facilitating barrier-free movement of disabled people and are used for short-distance transports and rehabilitation care for people with disabilities or patients. Using a lever that controls the elevation of the seat, nursing staff can easily implement patient transports.


It significantly reduces the work intensity of nursing staff and family members, improves care efficiency, reduces nursing risks and prevents patients from suffering secondary injuries during transports, while improving the quality of life and dignity of people with disabilities and promoting rehabilitation.

ATTENTION

Always observe the basic safety precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

Elevator construction and technical parameters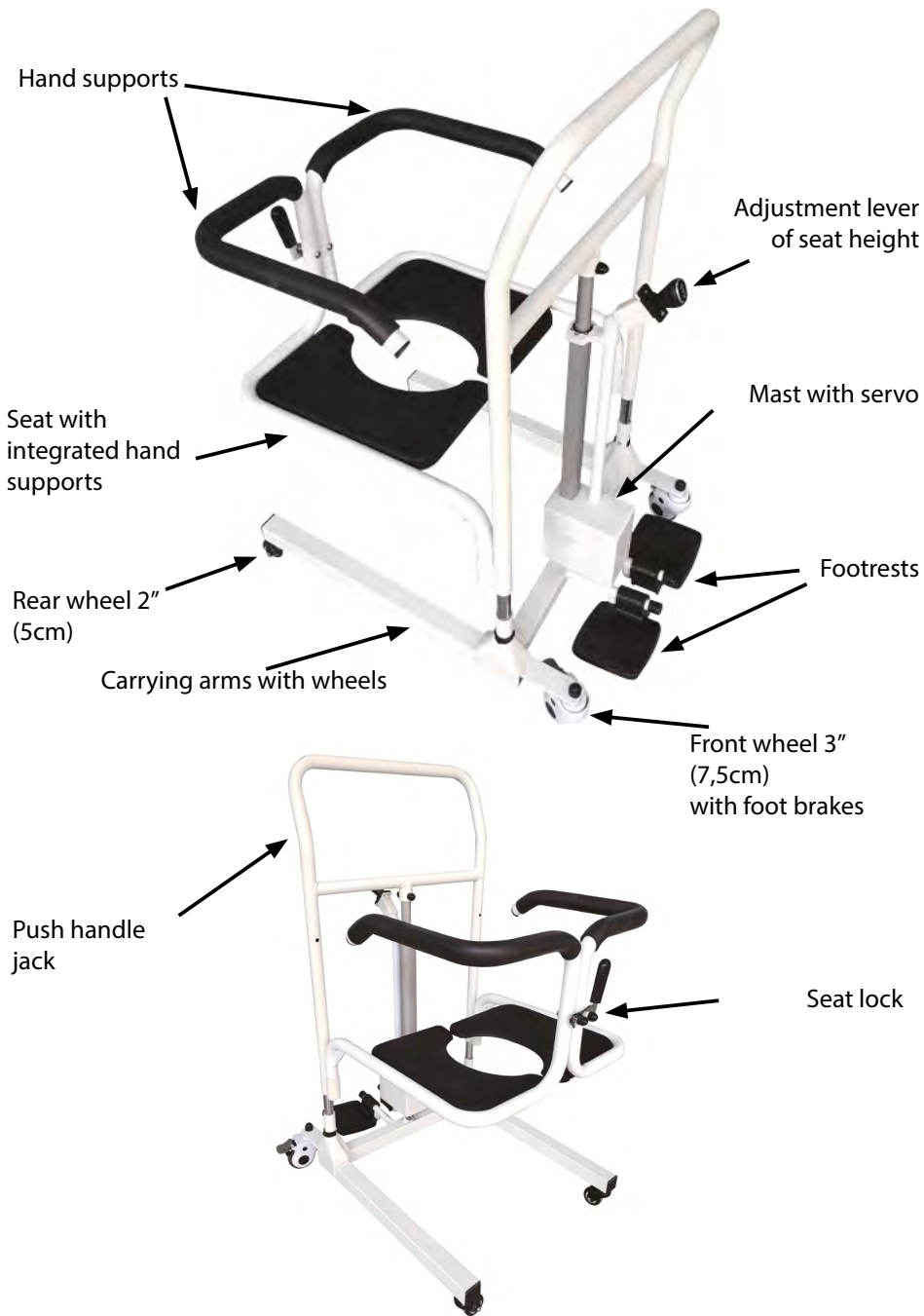

Technical specifications

Parameter	Value
Overall external length	75 cm
Overall external width	53 cm
Overall external height	125 cm
Inner seat width	44,5 cm
Inner seat depth	35 cm
Range of seat height adjustment	25 cm
Seat overrun height (measured from the ground)	minimum: 39 cm maximum: 64,5 cm
Travel arm height with castors	11 cm
Total height of arm rests	30 cm
Diameter of the front wheels	7,5 cm (3")
Diameter of the rear wheels	5 cm (2")
Weight of the lift	24,3 kg
Maximum patient weight	135 kg

Indications and contraindications

Indications

Assistive equipment is designed for people with mobility disabilities to facilitate (overrun over the toilet) or to a limited extent for indoor transportation of people with mobility disabilities.

Contraindications

Lack of independent stabilization of the head and trunk. Contraindications to sitting. Stroke, total paralysis. Patient weight over 135 kg.

Intended user

A medical device intended for use by a professional user.

Target patient groups

The described medical device is intended for people with mobility dysfunctions. Intended for people with mobility dysfunctions, facilitating (toilet seat overrun) or to a limited extent, transportation of the disabled indoors.


Assembly

After unpacking the package, check that it contains all the parts as listed below. Make sure that all parts are free from visible mechanical damage. If in doubt, contact the manufacturer. Do not use the unit until all parts have been properly assembled.

List of parts

S.n.	Part/element	Pcs.	Comments
1	Lower seat holder with wheels	1	
2	Tube rail locks	2	
3	Left-hand seat holder	1	
4	Right-hand seat holder	1	
5	Mast with servo	1	
6	Charging Adapter	1	
7	Toolkit	1	2 x wrenches 10-13
8	Instructions for use	1	Version PL EN

Scheme of detachable mains connection

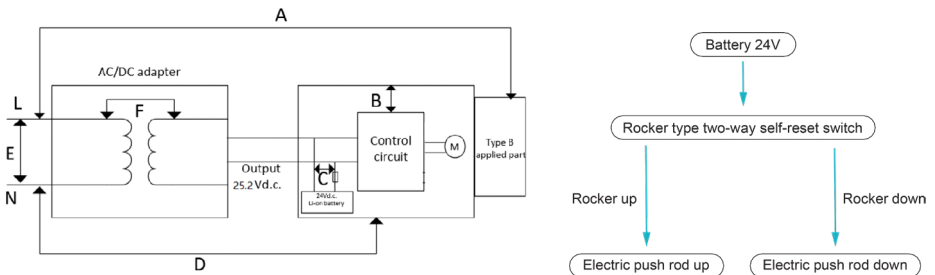WARNING: No modification of this equipment is permitted.

WARNING:


- Keep away from water jets
- Do not immerse in water
- Keep away from dust

Seat assembly


Assemble the guides.
Mount the tube guides and
secure them the pins
provided.

Mount the motor to the lower
part of the frame and lock
with the bolt with the sleeve.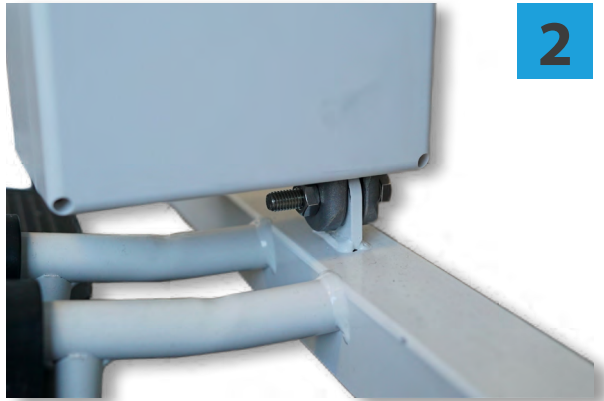

Mount the frame on
the tubular slides.

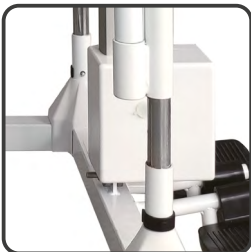Install and tighten the tube rail entry screw at the top of the servo.


4

Install the spring on the railing closing hook.

5

Attach the seat to the tubular handles.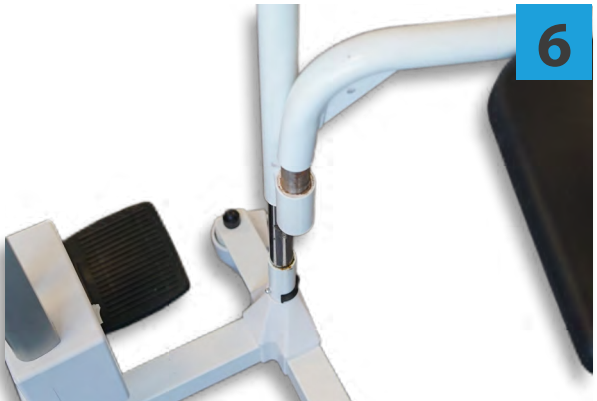

6

Usage

The lift is a non-sterile, reusable medical device.


Before using for the first time, make sure that all components are in working order and without signs of damage.

As shown in the images below, the hand supports integrated into the seat of the lift shall be provided with safeguards for the user. When transporting the user to the lift, the safety device must be released and the hand supports integrated into the lift seat should be movable so that they can be freely manipulated.


WARNING:
The seat overrun height is measured from the ground to the bottom edge of the seat - as shown in the photo opposite.

Overrun height:
minimum: 39 cm
maximum: 64,5 cm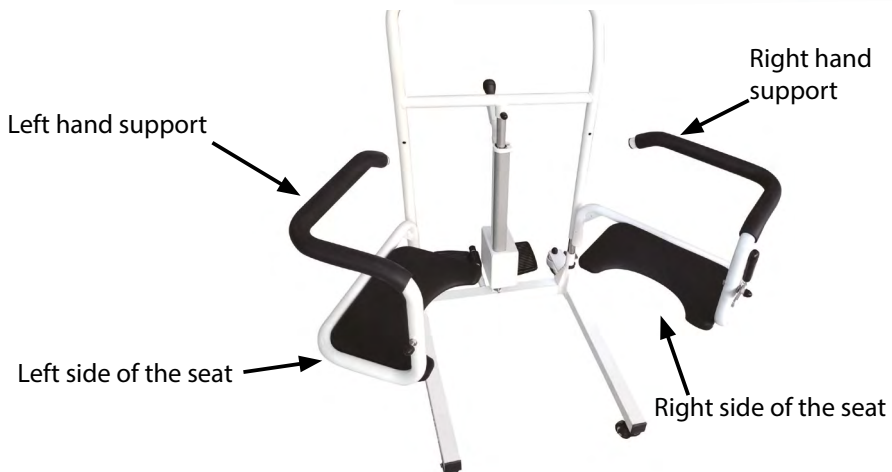


Patient transport to an lift - simulation

The following figure shows a simulation of a patient transport on a hoist. The lift with the restraints released has been brought to the patient, who is symbolized by the figure with the dotted line. Movable hand supports, integrated into the seat, make it possible to adjust the position of the lift to the patient's position without exposing the patient to undue difficulty and effort. During transport, it is possible that the patient's pelvis will need to be raised slightly so that the seat can be placed under the patient. Once the user is in a safe position on the lift, close the locking mechanism (seat lock). The lift must have the foot brakes applied each time the patient is transported.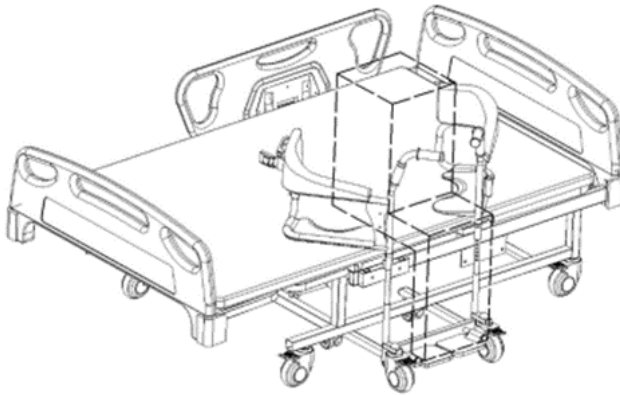

Patient transport to lift - different surfaces

The figures below show the correct positioning of the lift when transporting a patient to a given usable surface and when preparing the patient for physiological activities.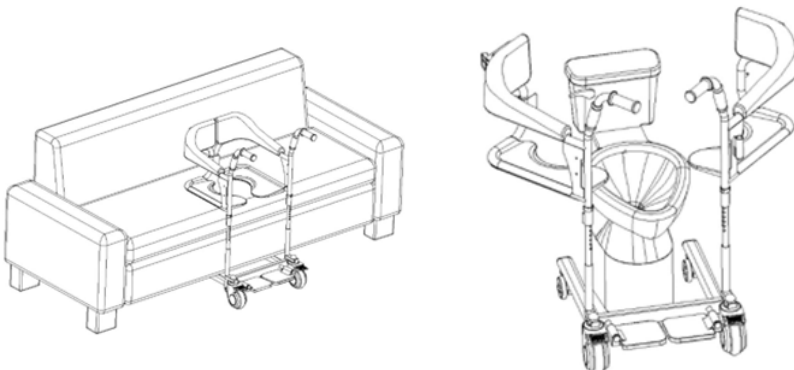

Short distance lift driving

The lift is designed for short distance patient transport. The person operating the lift should be trained in the use of the lift. For increased patient comfort, the lift is equipped with a seat cushion.


The lift must have the foot brakes applied for each transport. The brake must be released before driving the lift.

Lift seat height adjustment

Changing the position of the lever allows you to raise or lower the patient. Pay special attention to a correctly mounted stem (cylinder retaining screw).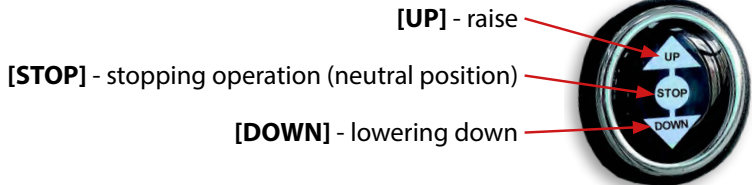

The servo will automatically stop when it is at its highest or lowest point. The electric motor operates on a low voltage supply, which has a great influence on safety.

User and patient safety

- a. Please read these instructions for use before using the lift and keep them for future reference
- b. Do not step on the footrests before getting on or off the patient lift.
- c. Lock the brakes when getting on or off the patient lift!
- d. Do not step on the footrests before getting on or off the patient lift.
- e. Do not move the lift with the user.
- f. The Lift is not intended to be used to transport a patient up stairs, as a stretcher or by hoisting the patient on it. The structure of the lift and the connecting parts are not designed for such high loads if used in this way. This may weaken or damage the entire load-bearing structure and cause an accident!
- g. The lift is designed for indoor use on paved and level surfaces only.
- h. Only use the lift on surfaces where all four wheels of the lift touch the ground.
- i. Do not leave the jack in the immediate vicinity of an open flame source.
- j. Please note that the surface temperature of the seat and frame parts may increase when exposed to the sun or other heat source, and there is a danger of very high cooling of the metal parts of the lift in cold weather. This can cause burns or frost-bite to body parts.
- k. The lift should be carried by a minimum of two people grasping the frame structure.
- l. Use extreme caution when navigating on inclined surfaces.
- m. The brakes are not intended to slow the lift during movement and should only be used to prevent unwanted movement of the lift.
- n. Do not use the lift if there is any visible or perceptible damage (e.g. breakage of the frame) as this may result in an accident.
- o. Before each use, perform a visual inspection of all structural components of the product, including:
 - check the operation of the braking system. Brake by pressing the brake with your foot.
 - carefully check the stability and condition of the support frame.
- p. All activities such as getting on, getting off, changing the patient's position, toileting activities **ABSOLUTELY** require:
 - constant assistance from the caregiver, the design of the lift does not give the patient the ability to steer and drive the lift, and to lock or unlock the brakes. - Locking the running wheel brakes.

- q. Do not leave the patient in the wheelchair (even with the brakes applied) on exits, ramps, mid-floors, etc.
- r. In order to maintain the performance of the lift, it is extremely important to keep all its elements clean. Storage of the elevator in very humid rooms (baths, laundries, etc.) or in the open air, combined with poor hygiene, will result in a rapid loss of the lift's functional and aesthetic parameters.
- s. Do not operate the lift if there is any visible or perceptible damage, or wear on any part of the lift which indicates a potential risk of accident (e.g., wear of running wheels, cracks, etc.). Refer all repairs to qualified personnel and, during the warranty period, only to the manufacturer's service department.
- t. When transporting (e.g. in a trunk), do not put any weight on the lift, it may affect its structure.
- u. Keep out of reach of children under five years of age.
- v. Refer all repairs to qualified personnel of the distributor or manufacturer.
- w. Use extreme caution when folding the lift, lack of attention may result in body parts getting crushed by the folding parts.

WARNING: Any "serious incident" involving a device needs to be reported to the manufacturer and to the competent authority of the EU Member State in which the user or patient resides.

WARNING: To avoid risk of electrical shock, the unit must be connected to a power supply with protective grounding only.

Maintenance operations

- Charge the battery before use.
- The battery should be recharged at least once a week or when discharged if this occurs first.
- Disinfect seat components after each use.
- Use toilet cleaners as recommended by their manufacturer's recommendations.
- Clean the elevator frame with a damp cloth and the seat with mild detergents.
- On an ongoing basis, remove dirt from the lift's moving parts (wheels, bolts, clamp locks).
- In order to maintain the functional parameters of the product it is extremely important to keep all its elements clean and to adjust to the way of its storage.
- Storage in rooms with very high humidity (baths, laundries, etc.) or in the open air, combined with poor hygiene, will result in a rapid loss of performance and aesthetics.

Repair

If repairs are necessary, contact your distributor.

Symbols

	Medical device
	Production date
	Manufacturer
	Authorized representative in the European Community / European Union
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	Lot code
	Unique Device Identifier
	Follow the instructions for use
	Refer to the instructions for use
	Warning
	Type BF application part
IP44	Protection against objects larger than 1mm Protection against splashing water from all directions

	<p>Possibility of jamming fingers</p>
	<p>Permitted temperature</p>
	<p>For indoor use</p>
	<p>Maximum patient weight</p>
	<p>Maximum safe inclination</p>
	<p>The product is designed to be transported by car</p>
	<p>The product is not intended to be used as a seat for transport in a motor vehicle</p>
	<p>We recommend short and thorough hand washing of the seat cushion at 30°C, do not use bleach, do not iron, do not dry clean, do not use stain removers containing solvents, do not use a tumble dryer</p>
	<p>DISPOSAL OF WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT</p> <p>This symbol on the products or accompanying documents means that used electrical or electronic products should not be disposed of with the normal household waste. For proper disposal, refurbishment or recycling, please take these products to the collection points for this type of waste.</p> <p>Alternatively, in some EU countries or other European countries, you may return your product to your local retailer when purchasing a similar new product. By disposing of this product correctly, you will help to conserve valuable natural resources and support the prevention of potential negative effects on the environment and human health, which could result from incorrect disposal.</p> <p>For further information, please contact your local municipality or collection point waste collection points. Penalties may be imposed in accordance with local regulations if this product is disposed of in an inappropriate manner. For parties within the European Union, If you wish to discard electrical or electronic equipment, please obtain the necessary information from your retailer or supplier.</p> <p>Elimination in countries outside the European Union. This symbol is valid in the European Union. If you wish to decommission this appliance, please obtain the correct decommissioning information from your local authorities or from your retailer.</p>

Contact information

XIAMEN SENYANG CO.,LTD
Address: No.189-1,Tongji South Road, Jimei District
Xiamen, 361000 Fujian P.R. China

SUNGO Europe B. V.
Address: Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Importer

mdh Sp. z o.o.
Address: Maratońska street 104, 94-007 Łódź, Poland
tel. +48 42 674 83 84, fax. +48 42 636 52 21
www.mdh.pl
www.viteacare.com

Notatki

Notes

mdh Sp. z o.o.
ul. Maratońska 104, 94-007 Łódź, Polska
tel. +48 42 674 83 84, fax. +48 42 636 52 21
www.mdh.pl
www.viteacare.com